

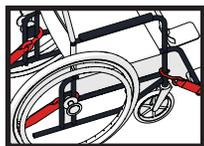
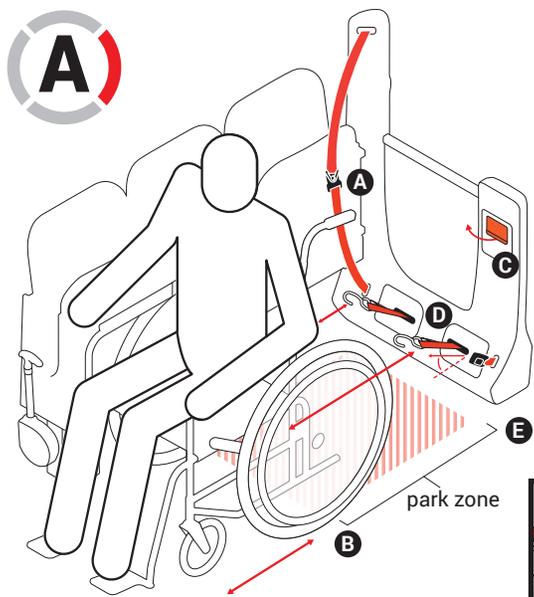


## ⚠️ WARNINGS



- Make sure to read, understand and follow all these instructions.
- J-Hooks must be attached to a solid wheelchair frame (no spokes or removable components) and **Rear J-Hooks** at approximate 30-45 degree angle with the floor.
- **Shoulder and Lap Belt** (we recommend using both) must go across occupant's shoulder and hips, and not be worn twisted or held away from the occupant's body by wheelchair components.

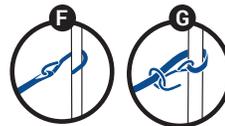

ex: J-hook tie-down

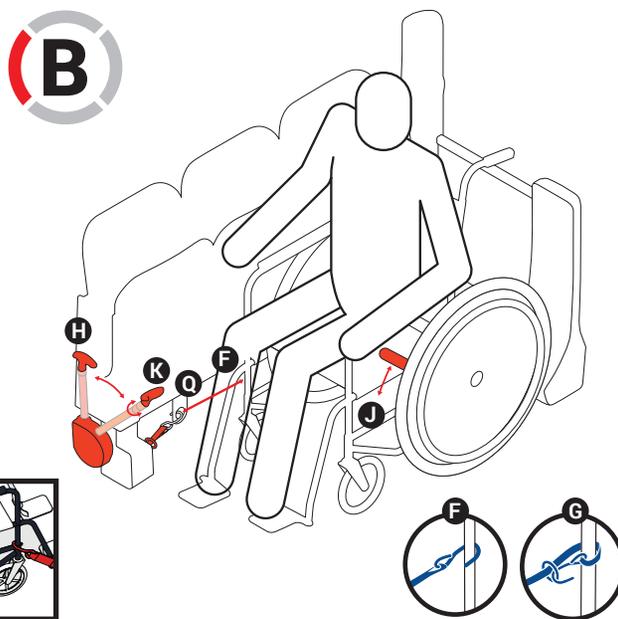

wheelchair scooter

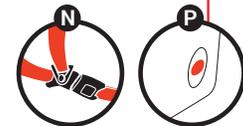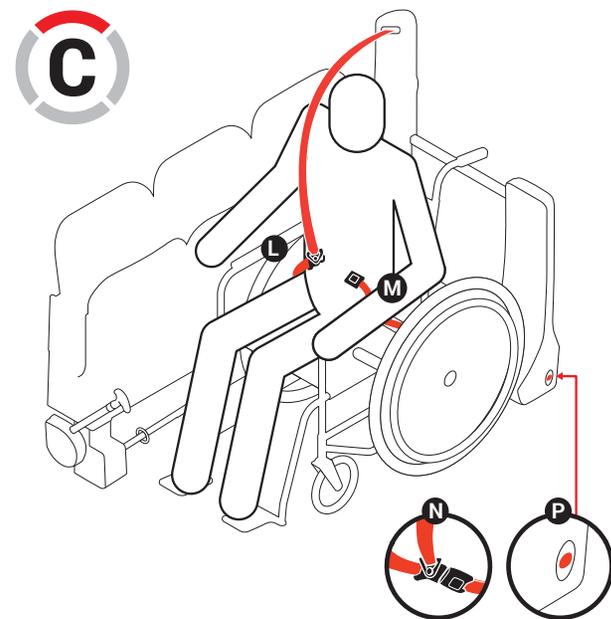

## SECURING

### A REAR

1. Attach shoulder belt pin connector to pin on wall side lap belt, then secure seatbelts onto the storage pin under flip-up seat (A).
2. Backup wheelchair to park zone (B).
3. Pull paddle handle (C) to release rear retractors and activate XPress Timer.
4. Attach both rear J-hooks to rear wheelchair frame (D).
5. Backup wheelchair within park zone (E) and against side bumper or closest proximity.

### B FRONT

6. Attach front J-hook to front wall side wheelchair frame (F). For **scooters**, loop J-hook around front wheel frame, and then insert J-hook (facing out) through O-ring (G).
7. Pull down tension handle (H) to tension front J-hook (F).
- ⚠️ Do not over tension J-hook.
8. Lock wheels (J) or power off chair.

### C SEATBELTS

9. Extend shoulder belt and wall side lap belt across occupant's shoulder and hip (L); then buckle them to the aisle side lap belt (M).
- ⚠️ Buckled seatbelts should be against occupant's hipbone (N).
10. Occupant and wheelchair are secured and ready to go.

## RELEASING

### C SEATBELTS

1. Detach shoulder belt pin connector from pin on lap belt (L).
2. Unbuckle lap belt (M) and place all seatbelts in storage.
- ⚠️ If needed, press seatbelt release button (P).

### B FRONT

3. Pull down (H) and turn tension handle (K), then return tension handle to the vertical position (H) to release front J-hook tension (F).
4. Remove front J-hook from wheelchair frame (F) or scooter (G). Place J-hook in storage ring (Q).
5. Unlock wheels (J) or power on chair.

### A REAR

6. Pull paddle handle (C) to release rear retractors and activate XPress Timer.
7. Move wheelchair forward, remove rear J-hooks from rear wheelchair frame and place J-hooks in storage (D).
8. Occupant and wheelchair are released and ready to go.